The over bed pole is a medical device and an aid for people, who cannot sit up in or stand up from bed safely without assistance. Residual forces of the upper body and the arms are required.

Technical Requirements: The over bed pole can only be used for beds with open underframe, not for closed headboards!

Indication: Bedridden patients, restriction of the lower extremities

Contraindication: back injury, apoplexy, hemiplegia, instable (vertebral) injuries

Description: The over bed pole is used to sit up in bed without help, to raise the buttock for using a bed pan, to turn over in a wheelchair or toilet chair etc. The product must only get into contact with intact, unwounded skin. Other applications than those listed are considered not to be in accordance with the intended use and are not permitted. Unauthorized modifications, incorrect usage and incorrect installation exempt the manufacturer from liability for any damages resulting therefrom. For damages resulting therefrom no liability by Sundo GmbH, the risk is borne exclusively by the user. Any serious incidents arising in connection with this product must be reported to the manufacturer or the competent authority without delay.

Before use / maintenance: Read the manual carefully and completely, keep it until the end of the product life. Please check all parts for (shipping) damage by visual inspection before first and every use. Movable parts must be regularly checked for smooth function. At least monthly review of the bed pole, especially welding seams and handle. Regular check of the belt strap, defective parts have to be replaced immediately. Replacement at the latest after 5 years.

Before each use, check that the strap and handle are firmly installed and undamaged!

Usage:

- ▶ The over bed pole has to be placed detached at the end of the head of the bed. The carrying arm must be placed parallel to the bed.
- ▶ Before each use the caregiver has to check the correct fitting of the strap.
- ▶ The handle has to be adjusted to the appropriate height, it should be installed at the height of your chin / chest while sitting.
- ▶ The over bed pole must only be loaded vertically.
- ▶ Do not move from side to side, tipping hazard!
- ▶ To turn over in a wheelchair or toilet chair a caregiver must provide assistance to the patient.

Cleaning / disinfection: All parts are best cleaned with a damp cloth, using mild detergents. Do not use any abrasive or sharp objects or cleaning supplies. For the care of the plastic parts we recommend a special plastic cleanser. The bed pole can be disinfected with a commercial, chlorine-free disinfectant. **Important:** When using disinfectants, the instructions for use of the manufacturer must be observed. **Handle and strap have to be replaced in every re-use / change of patient!**

Information for re-use: the over bed pole must be cleaned and subjected to maintenance each time before re-using, the MPG guidelines (Medical Devices Directives) are to be complied with. Cleaning should be carried out in accordance with the guidelines from the Robert Koch Institute (RKI) for hospital hygiene and the prevention of infection.

Maintenance: Monthly review of the bed pole, especially welding seams and handle. Please check the belt strap regularly, defective parts have to be replaced immediately.

! Safety Instructions:

- ▶ The product must only get into contact with intact, unwounded skin.
- ▶ Please check the traction of strap and handle strongly before using the bed pole
- ▶ Note the maximum load / traction (120 kg).
- ▶ Do not move from side to side, tipping hazard!
- ▶ The bed pole must be placed parallel to the bed, not crosswise!
- ▶ Assembly only as shown!
- ▶ The bed pole must only be used on flat ground!
- ▶ If parts are warped, have cracks or deformations, the bed pole must not be used any longer!
- ▶ The bed pole is no toy! Do not leave children unattended with the bed pole!
- ▶ Transportation only disassembled!
- ▶ Do not fix other things or objects at the over bed pole!

Warranty: According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly. We reserve the right of modifications serving the technical improvement.

Sundo Homecare GmbH,
Bahnhofstraße 13, DE-30916 Isernhagen, www.sundo-homecare.de

Valentine International Ltd., 8th Floor, No. 149, Sec 2, Ta Tung Road,
221 Hsichih District, New Taipei City, Taiwan R.O.C.

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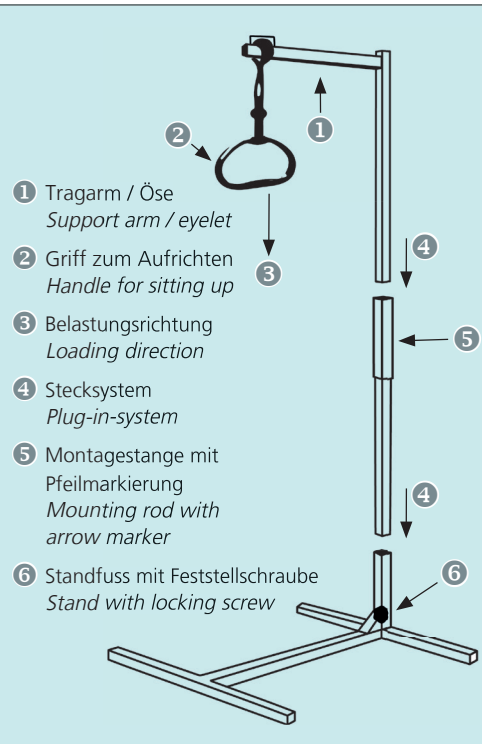
Assembly:

The over bed pole is delivered preassembled. You can install it without any tool, simply placing the square pipes into each other (see illustration). Then tighten the screw on the vertical hold of the stand to fix the mounting rod. Attach the handle to the end of the support arm by pulling the strap through the eyelet and setting the required length. Finally check the correct and firm fit of the strap / handle by pulling it.

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