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## Instructions of use for impression foam

### 1. Suggested use:

Production of negative footprint in the orthopaedic field.  
Handling to be done only by skilled staff.

### 2. Storage:

Impression foam shall be stored only in dry places protected from humidity.

### 3. Application:

- Use Impression foam only in dry condition.
- Unnecessary long skin contact has to be avoided.
- Direct contact with wounded skin has to be avoided. Wounded skin has to be covered and protected by thin - and if necessary - by sterile foil.
- Direct contact with mucous membrane has to be avoided. In case of direct contact immediately clean with water.
- Impression foam not suitable to eat or swallow.
- Impression foam shall be used only one time.

### 4. Disposal:

Impression foam can be disposed like normal rubbish but separation of plastic (foam) and paper is recommended. Contaminated impression foam (e.g. microbial contamination with secretion) must be treated like medical waste.

### 5. Information

Our company is certified under DIN ISO 9001:2008 Our Impression foam is checked after German Institute for Standardization EN ISO 10993-5 and German Institute for Standardization EN ISO 10993-10. During the usual opening hours, we are glad to be of your assistance for questions and suggestions

### AT Kunststoffe GmbH

Bocholder Esch 31 a  
D-48683 Ahaus-Alstätte  
Tel.: (0 25 67) 93 78 70  
Fax: (0 25 67) 93 77 02  
E-Mail: info@atkunststoffe.de